
Endexo® surface modification

Reduce thrombus and improve blood compatibility on device surfaces

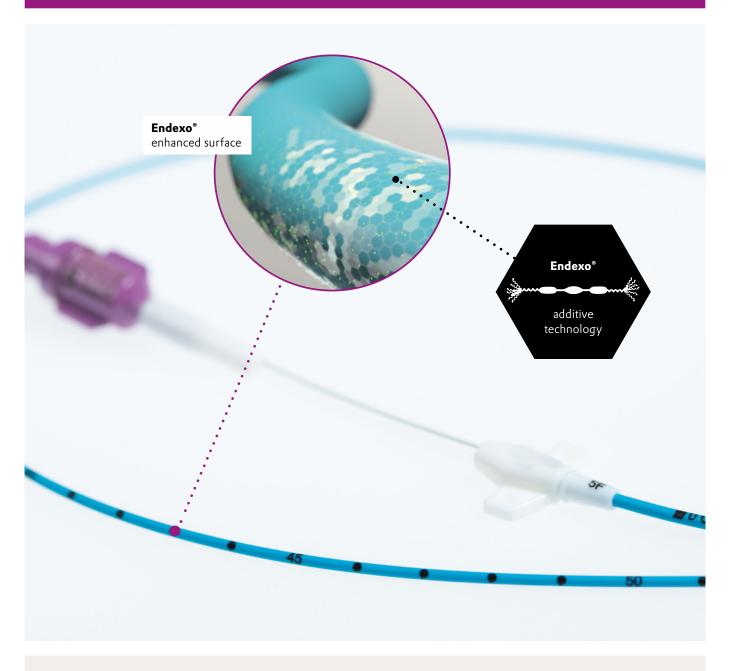

Key benefits of Endexo®

1

Passive surface

2

Compatible with range of polymers

3


Proven performance



Endexo® uniformly modifies all device surfaces to reduce risk of thrombosis

Endexo® additive technology can modify the surfaces of blood-contacting devices to reduce adhesion and activation of blood components, thereby significantly reducing thrombus formation and associated complications. Endexo® additives are blended with the medical device polymer during device manufacturing, and uniformly modify all surfaces with no impact on device functionality. *In vitro* blood loop studies show **up to 99%** reductions in thrombus formation on Endexo®-modified devices compared to unmodified devices or commercial products.

Thrombus adhesion on Endexo®-modified devices relative to unmodified Controls in 2 hour *in-vitro* bovine blood loop model

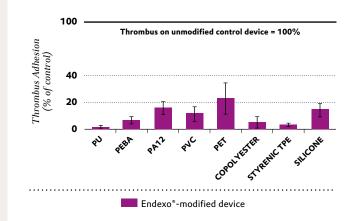

- Compatible with range of polymers and manufacturing processes
- Requires no post treatment or secondary application steps
- Passive mode of action (no biologicals or drugs)
- Durable and does not elute or leach
- Suitable for complex geometries and small profiles

Unmodified PU Tubing




Endexo®-modified PU Tubing

Proven clinical performance

Endexo® has been used in medical devices for over a decade, with a track record of clinical benefits in cardiovascular and neurovascular applications. Endexo® offers a simple pathway for development, manufacturing, and market approval of your device.

- ¹ Chopra V et al. Lancet 2013: 382: 311–25. Results may vary.
- ² McDiarmid S et al. CMAJ Open 2017; 5(3): E535-E539. Results may vary.
- ³ Integra Lifesciences CerebroFlo® brochure, 2020.

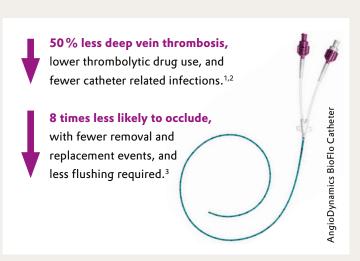

This information and all further technical advice are based on our present knowledge and experience. However, it implies no liability or other legal responsibility on our part, including with regard to existing third party intellectual property rights, especially patent rights. In particular, no warranty, whether express or implied, or guarantee of product properties in the legal sense is intended or implied. We reserve the right to make any changes according to technological progress or further developments. The customer is not released from the obligation to conduct careful inspection and testing of incoming goods. Performance of the product described herein should be verified by testing, which should be carried out only by qualified experts in the sole responsibility of a customer. Reference to trade names used by other companies is neither a recommendation, nor does it imply that similar products could not be used.

Evonik Operations GmbH Health Care Business Line

healthcare@evonik.com www.evonik.com/healthcare

